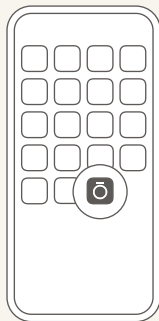
Stuff you
should *know*

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
Get connected

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

1


2

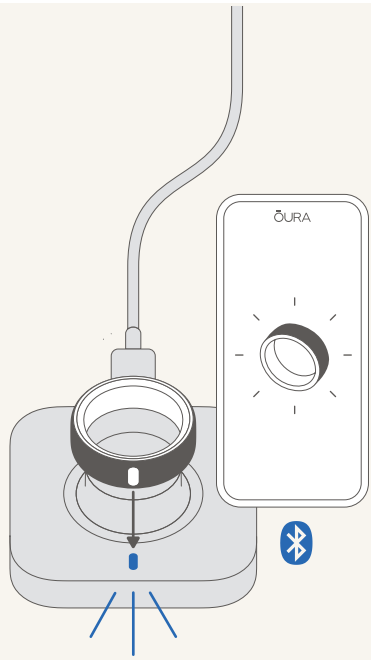3

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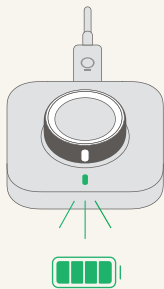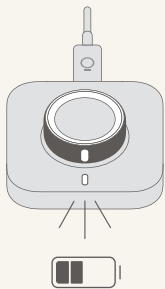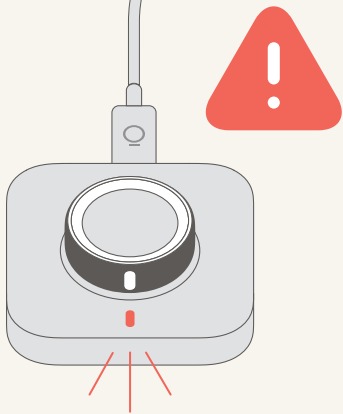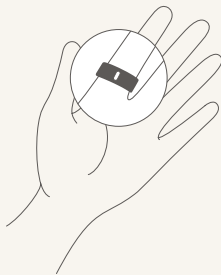
100m

Sauna

Oura Ring 4 is designed for 24/7 use, and can be worn anywhere you go. Oura Ring is water-resistant up to 100 m / 328 ft.

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Wear your Oura Ring with the sensors facing the palm of your hand.

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Clean your Oura Ring regularly with soap and water. Keep it dry to ensure good functionality and reduce potential skin irritation.

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Questions? Refer to the Frequently Asked Questions (FAQs) or contact support at support.ouraring.com. You can also contact us on your Oura App > Home > Help.

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English

Thank you for choosing Oura Ring 4!

We want to do our best to support you on your health journey with Oura Ring. Here you'll find information about Oura Ring 4 and the Oura Ring charger. This booklet lists the technical details and safety information of Oura Ring. Please read through carefully to use Oura Ring safely and get the best experience with ÖURA.

General usage

Oura Ring 4 is designed for 24/7 usage, and can be worn during showers, saunas, baths, swimming, and snorkeling. **Oura Ring is water resistant to 100 m / 328 ft.**

△ Clean the ring regularly and keep it dry, to ensure good functionality and reduce potential skin irritation. You can clean the ring by wiping down with a damp cloth and drying the ring fully before using again.

△ Use the provided size-specific Oura Ring 4 charger and cable for charging.

△ **Oura Ring charger is not water resistant** and is intended for indoor use only. Do not use or store the charger in humid environments like bathrooms.

△ The Oura Ring may get scratched and can itself scratch softer materials (metal, jewelry or other objects) while using the ring in close contact.

Safety

Oura Ring is not a medical device, and is not intended to mitigate, prevent, treat, cure or diagnose any disease or condition. Always consult a physician regarding medical conditions. Always be sensitive to your body's responses and if you make any changes to your behavior based on data from your Oura Ring, you agree that you do so at your own risk.

△ **If you feel unexpected, repeated, or long-term pain, fatigue, or discomfort due to having made changes to your sleep or activity, consult a physician before continuing with such changes.**

△ **Ring surface materials are safe for skin contact (ISO 10993 standard). If you experience irritation on your skin due to the Oura Ring, remove the ring.** If symptoms persist after not using Oura Ring, please contact a physician.

△ Please be careful that the Oura Ring does not get caught on fixed structures or heavy objects.

△ **In cases of emergency or discomfort, or if you can't remove the Oura Ring yourself, seek immediate medical attention.**

△ **Keep Oura Ring away from children.** This product is not intended for individuals under the age of 18.

△ **Do not expose Oura Ring or the charger to heat, such as in a vehicle or directly in the sun. Do not charge Oura Ring in such an environment.**

△ **Do not attempt to open, cut or puncture the Oura Ring.**

△ **Avoid handling batteries, working on devices with machinery that contain batteries, or situations where there is a risk of electric shock.**

Product Information

→ **Operating temperature range** -10 – 54 °C / 14–129 °F

→ Red and green light-emitting diodes (LED) in the product are in the visible color range. They are safe for the wearer and other people nearby.

→ **Memory capacity** Recommended to sync Oura Ring data daily with Oura App. Oura Ring is capable of storing up to 2 weeks of data depending on the type and frequency of ring use.

→ **Battery** Rechargeable Li-ion battery.

→ **Battery life** Typically up to 8 days. Actual battery life depends on Oura Ring size, usage, Oura App settings, age, and many other factors.

→ **Charging time** Typically charges fully in 20-80 minutes, depending on the starting battery level

→ **Charging** at room temperature. Optimal charging temperature: min 18 °C / 65 °F - max 32 °C / 89 °F. Normal operation distance for Charger model OA12 is more than 20 cm (8 in) from the human body.

→ **Charger and power cord** USB Type-C connector in charger. Power cord type USB-C to USB-C. Use external 5Vdc power supply that complies with IEC 62368-1 standard.

→ **Oura charger complies** with IEC 62368-1 international safety standard for ICT and AV equipment.

→ **Connectivity** Bluetooth Low Energy

→ **Ring firmware updatability** Automatic firmware updates via Oura App

→ **Oura App availability** App Store, Google Play

→ **Oura App compatibility** See ouraring.com/compatibility

→ **Security** Encrypted Bluetooth communication used between Oura Ring and Oura App

→ **Dispose of Oura Ring** (and internal battery) in accordance with applicable local laws and regulations. Special handling may be required. The charger is an electronic device and local legislation for electronics recycling should be followed.

→ **Factory reset instructions** at support.ouraring.com to perform a factory reset before ring disposal.

Warranty Disclaimer: ouraring.com/terms-and-conditions

Support and FAQs

If you have questions about using your Oura Ring, Oura App, or our online services, please do not hesitate to contact us.

You'll find Frequently Asked Questions (FAQs) at support.ouraring.com.

You can contact Support at support@ouraring.com or on your Oura App: Home > Help.

You can access ÖURA privacy policies at ouraring.com/privacy-policy.

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English

Warranty Disclaimer

THE SERVICES ARE PROVIDED TO YOU "AS IS WITH ALL FAULTS" AND "AS AVAILABLE" WITHOUT WARRANTY OF ANY KIND, AND OURA AND ITS SUBSIDIARIES, AFFILIATES, OFFICERS, DIRECTORS, EMPLOYEES, REPRESENTATIVES, AGENTS, PARTNERS AND LICENSORS HEREBY DISCLAIM ALL WARRANTIES AND CONDITIONS WITH RESPECT TO THE SERVICES WHETHER EXPRESS OR IMPLIED AND EXPRESSLY DISCLAIMS ANY IMPLIED WARRANTIES OF TITLE, MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, SATISFACTORY QUALITY, AND NON-INFRINGEMENT. OURA DOES NOT WARRANT AGAINST INTERFERENCE WITH YOUR ENJOYMENT OF THE SERVICES, THE AVAILABILITY OF CONTENT, THAT THE FUNCTIONS CONTAINED IN THE SERVICES WILL MEET YOUR REQUIREMENTS, THAT THE SERVICES WILL BE FREE OF VIRUSES OR OTHER HARMFUL COMPONENTS, THAT THE OPERATION OF THE SERVICES WILL BE UNINTERRUPTED OR ERROR-FREE, THAT DEFECTS IN THE SERVICES WILL BE CORRECTED, OR THAT THE FUNCTIONS CONTAINED IN THE SERVICES WILL FUNCTION WITH OTHER MOBILE APPS OR HARDWARE, OR WITHIN A SYSTEM. NO ORAL OR WRITTEN INFORMATION OR ADVICE GIVEN BY OURA OR AN OURA AUTHORIZED REPRESENTATIVE SHALL CREATE A WARRANTY. SOME JURISDICTIONS DO NOT ALLOW THE EXCLUSION OF IMPLIED WARRANTIES OR LIMITATIONS ON APPLICABLE STATUTORY RIGHTS OF A CONSUMER, SO THE ABOVE EXCLUSION MAY NOT APPLY.

NOTWITHSTANDING THE FOREGOING, OURA WARRANTS TO THE ORIGINAL END USER PURCHASER THAT THE PRODUCT IS FREE FROM DEFECTS IN MATERIAL AND WORKMANSHIP FOR ONE (1) YEAR FROM THE DATE OF PURCHASE. If your local law in effect at the time of purchase requires a warranty period longer than one (1) year, this warranty shall be extended to the extent required by such law. Within the warranty period, Oura shall repair or replace at no charge to you any components of the Product that fail the out of warranty period. You shall be responsible for any related transportation charges. Replacement products may be new or refurbished at our discretion. This limited warranty does not apply to (i) normal wear and tear, including scratches and dents; (ii) consumable parts included in the Product, such as batteries, unless product damage has occurred due to a defect in materials or workmanship; (iii) damage resulting from your failure to use the Product in accordance with the instructions accompanying the Oura Product or available at the website; (iv) damage resulting from an accident, flood, fire, misuse, or abuse; (v) damage resulting from service performed, or damage resulting from tampering or alteration of the Product not authorized by Oura; or (vi) use of the Product with any other application or software than the Mobile App.

Oura retains the exclusive right to repair or replace the Product, or offer a full refund, at its sole discretion. Such remedy shall be your sole and exclusive remedy for any breach of this limited warranty. Warranty repairs and replacements have a new warranty which is the longer of ninety (90) days or the balance of the original one (1) year warranty.

Limitation of Liability

IN NO EVENT WILL OURA, ITS SUBSIDIARIES, AFFILIATES, OFFICERS, DIRECTORS, EMPLOYEES, REPRESENTATIVES, AGENTS, PARTNERS AND LICENSORS, BE LIABLE FOR DAMAGES OF ANY KIND, UNDER ANY LEGAL THEORY, ARISING OUT OF OR IN CONNECTION WITH YOUR USE, OR INABILITY TO USE, THE SERVICES OR ANY CONTENT ASSOCIATED WITH THE SERVICES, OR SUCH OTHER SITES OR ANY SERVICES OR

ITEMS OBTAINED THROUGH THE SERVICES, INCLUDING ANY DIRECT, INDIRECT, SPECIAL, INCIDENTAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, INCLUDING BUT NOT LIMITED TO, PERSONAL INJURY, PAIN AND SUFFERING, EMOTIONAL DISTRESS, LOSS OF REVENUE, LOSS OF PROFITS, LOSS OF BUSINESS OR ANTICIPATED SAVINGS, LOSS OF BUSINESS OPPORTUNITY, BUSINESS INTERRUPTION, LOSS OF USE, LOSS OF GOODWILL, LOSS OF DATA, AND WHETHER CAUSED BY TORT (INCLUDING NEGLIGENCE), BREACH OF CONTRACT OR OTHERWISE, EVEN IF FORESEEABLE. THESE EXCLUSIONS OR LIMITATIONS WILL APPLY REGARDLESS OF WHETHER OR NOT OURA HAS BEEN WARNED OF THE POSSIBILITY OF SUCH DAMAGES.

THE FOREGOING DOES NOT AFFECT ANY LIABILITY WHICH CANNOT BE EXCLUDED OR LIMITED UNDER APPLICABLE LAW. TO THE EXTENT LIABILITY CANNOT BE EXCLUDED OR LIMITED AS SET FORTH ABOVE, IN NO EVENT SHALL OURA BE LIABLE FOR ANY CLAIM, WHETHER IN CONTRACT, TORT, OR UNDER ANY OTHER THEORY OF LIABILITY, IN EXCESS OF \$100.

Terms of Use

Your use of the Oura Products and Services is subject to our Terms of Use, available at ouraring.com/terms-of-use/, and Privacy Policy, available at ouraring.com/privacy-policy/. The Warranty Disclaimer and the Limitation of Liability set forth above are from our Terms of Use and are subject to change if we change our Terms of Use.

Intellectual property rights

OURA, OURA, and Ö are trademarks of Oura Health Oy. Oura Content, Oura Products, Oura Features and Services, and underlying technology are protected by copyright, trademark, patent, and other laws of the United States and foreign countries. All rights reserved. Manufactured for Oura Health Oy (Oura), Elektronikkatie 10, FI-00900 Oulu, Finland. Visit ouraring.com/intellectual-property-notice for a detailed listing of Oura's intellectual property rights.

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2014/53/EU

In order to satisfy the essential requirements of 2014/53/ EU Directive, the product is compliant with the following standards:

RF spectrum use EN 300 328 v2.2.2 and EN 303 417 V1.1.1

EMC	EN 301 489-1 V2.2.3 EN 301 489-3 V2.3.2 EN 301 489-17 V3.2.4 EN 55011:2016/A:2020 EN IEC 61000-6-1:2019 EN IEC 61000-6-3:2021
-----	----------------------------------------------------------------------------------------------------------------------------------------------

Health & Safety (RED art. 3.1a)	EN 62368-1:2018 EN 62479:2010 EN 62311:2008
---------------------------------	---------------------------------------------------

The conformity assessment procedure of Directive 2014/53/ EU has been followed with the involvement of the following Notified Body: DEKRA Testing and Certification, S.A.U. Parque Tecnológico de Andalucía C/ Severo Ochoa nº 2 & 6 29590 Campanillas – Málaga SPAIN

This product is compliant with Directive 2012/19/EU. Please dispose of used electrical and electronic equipment (WEEE) according to national laws and regulations with electrical waste. It should not be mixed with general household waste.

Regulatory notices

USA

Cura Health Oy has not approved any changes or modifications to these devices by the user. Any changes or modifications could void the user's authority to operate the equipment.

FCC regulatory information

FCC statements are available at the following internet address: curing.com/fcc-compliance-statements. Devices (FCC ID: 2AD7V-OURA2401, FCC ID: 2AD7V-OURA2402) comply with Part 15 and Part 18 of the FCC Rules. Operation is subject to the following two conditions: (1) devices must not cause harmful interference, and (2) devices must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 and Part 18 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause regulatory harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

RF exposure safety

This product is a radio transmitter and receiver. It is designed not to exceed the emission limits for exposure to radio frequency (RF) energy set by the Federal Communications Commission. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Canada

Innovation, Science and Economic Development Canada (ISED) regulatory information.

Devices (CAN ICES-1 (B)/NMB-1(B), CAN ICES-3 (B)/NMB-3(B); IC: 20635-OURA2401, IC: 20635-OURA2402) comply with the Interference-Causing Equipment Standards and Innovation, Science and Economic Development Canada's license exempt RSS standard(s). Operation is subject to the following two conditions:

- 1. Devices may not cause interference; and

2. Devices must accept any interference, including interference that may cause undesired operation of the device.

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RF exposure safety

This product is a radio transmitter and receiver. It is designed not to exceed the emission limits for exposure to radio frequency (RF) energy set by the ISDE.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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CAN ICES-1 (B)/NMB-1(B), CAN ICES-3 (B)/NMB-3(B)

This Class B digital apparatus complies with Canadian ICES-001 and ICES-003.

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OURA、OURA、Öura Health Oyの商標です。OURAのコンテンツ、OURA製品群、OURAの機能およびサービス、その基礎となる技術は米国およびその他の国の著作権、商標、特許、およびその他の法律により保護されています。All rights reserved. (すべての権利を留保します) 製造: OURA Health Oy (OURA)、Elektronikkaitie 10, FI-90500 Oulu、Finland、OURAの知的財産権に関する情報は、ouraring.com/ja/intellectual-property-notice/をご確認ください。
ŌURA, ŌURA 및 Ō는 Ōura Health Oy의 등록상표입니다. Ōura 콘텐트, Ōura 제품, Ōura 기술 및 서비스와 기반 기술은 미국 및 다른 국가의 저작권, 등록상표, 특허 및 기타 법률의 보호를 받습니다. All rights reserved. Ōura Health Oy(Ōura)(주:스:Elektroniikkatie 10, FI-90590 Oulu, Finland)를 위해 제조되었습니다. Ōura 지식재산권 상세 목록은 ouraring.com/intellectual-property-notice에서 확인할 수 있습니다.